Tubos Químicos Chemical Tubes

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Chemical Tubes CD 210/220/230/240/250 For sterilization control

English

The chemical tubes contain a heat sensitive red liquid which changes color from red to green at various times and temperatures that are integrated by the device. The chemical tubes are calibrated to respond to specific temperature and time parameters during a STEAM or DRY HEAT sterilization cycle. They have a permanent colour change when the proper parameters are achieved at the location of each control tube. A color guide in the instruction provides an instant visual check of the results. Since the chemical tubes consist of sealed vials containing heat-sensitive solution, chemicals from the indicator do not normally come into contact with adjacent items during the cycle.

The liquid in the unheated tubes is RED. Sterilization proceeds, the liquid changes colour from red to GREEN. If, after treatment, the liquid remains red or amber the materials should be re-sterilized and a fresh tube must be used.

The chemical tubes do not ensure proper sterilization of the product. They only allow to show if critical parameters have been integrated in the sterilization process for which they were designed.

Sterilization by Steam

Two types of tubes are available:

BLACK SPOT: For use in steam and fluid sterilizers working at 121 °C for 15 minutes or 126 °C for 10 minutes.

YELLOW SPOT: For use in steam autoclaves working at 134 °C for 3 to 3.5 minutes.

Handling information

Place the tube in the centre of the drum or package to be sterilized.

Sterilization by Dry Heat

GREEN SPOT: For dry heat ovens/sterilizers working at 160 °C for 60 minutes or 170 °C for 30 minutes.

BLUE SPOT: For dry heat conveyor ovens at 180 °C for 12 minutes.

WHITE SPOT: For dry heat ovens/sterilizers working at 160 °C for 120 minutes, 170 °C for 60 minutes or 180 °C for 35 minutes.

Instructions for use

A chemical tube should therefore be put in the middle of the batch of apparatus to be sterilized and should be enclosed in a similar container to that used for the apparatus. For example, if syringes are being sterilized inside test tubes then the chemical tube should be placed inside a similar test tube similarly closed. It is indeed an advantage to put the chemical tube inside one of the objects to be sterilized (for example, where possible inside the barrel of one of the syringes) to ensure that heat has penetrated to the innermost parts.

IMPORTANT: Excessive exposure to high temperatures/long times may result in fading of colour. Do not use above 200 °C. If used at temperatures below 140 °C in Dry Heat this may not give the requested level of sterility assurance.

Caution

For all cases, after the sterilization process, handle tubes with gloves or tweezers and glasses. Cooling the product before handling.

Not shake.

Not use for other sterilization processes.

Not brake the glass tube.

Storage

These chemical tubes must always be stored in a cool place, preferably in a refrigerator, but always below 21 °C, otherwise they may deteriorate.

Processed tubes may be stored for record purposes in cool, dark conditions, or disposed of as glass waste.

Color guide

Provides an instant visual check on your results. However, references colors may not be accurate.

Industria Argentina - Made in Argentina

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Fabricado por: Terragene S.A. - Güemes 2879 - (2000) Rosario - Santa Fe - Argentina