

Precintos contenedores

CS20/50

Español

Precintos contenedores para control de procesos de esterilización por Vapor.

Condiciones: 10 minutos 121 °C; 2 minutos 134 °C.

Color inicial: **azul**

Color final: **negro**

Descripción de producto

Los Precintos Contenedores están formados por una etiqueta autoadhesiva con indicador químico y un cinto de 7.5 cm de longitud.

El área de la etiqueta proporciona el espacio necesario para registrar la fecha de producción y de caducidad, número de lote, las iniciales de la persona responsable de la liberación de la carga y el contenido de los contenedores.

Los Precintos contienen un indicador de proceso Clase 1 para Vapor de acuerdo a la norma EN ISO 11140-1: 2009.

Aplicación

El producto sirve para sellar instrumentos contenedores y así prevenir el uso accidental del contenido en caso de que el contenedor haya sido abierto por error o no haya sido esterilizado.

El cinto debe insertarse en la construcción del Precinto y la punta del mismo queda adherida en la etiqueta plegada.

Almacenamiento


Mantener los indicadores en lugares secos, al abrigo de la luz y a una temperatura entre 15 - 30 °C, humedad relativa entre 35 - 60 %.

No mojar.

No almacenar cerca de agentes esterilizantes.

No utilizar los indicadores después de su fecha de vencimiento.

No reutilizar.

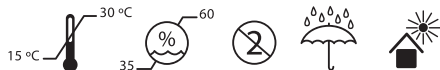

Container seals

CS20/50

English

Container seals for Steam sterilization processes.

Conditions: 10 minutes 121 °C ; 2 minutes 134 °C.

Initial color: **blue**

Final color: **black**

Product description

Container Seals are composed of a self-adhesive label with a chemical indicator and a pin of 7.5 cm length. The label area provides the space for production and expiry date, batch number, initials of the person responsible releasing the load and the container content.

The seal contains a steam Class1 process indicator according to EN ISO 11140-1: 2009 standard.

Application

The product is used to seal instrument containers to prevent accidental use of the contents if the container has been tampered with or opened by mistake or has not been sterilized.

The pin is inserted into the Container Seal construction and the end of the pin is adhered between the folded label.

Storage

Indicators should be stored in dry places, protect from light at temperatures between 15 - 30 °C, 35 - 60 % relative humidity.

Do not wet.

Do not storage close to sterilizing agents.

Do not use indicators after their expiration date.

Do not reuse.