# Indicadores Integradores

## Indicator Integrators

# Indicadores Integradores

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According to EN ISO 11140-1:2009 standard

Lic. Adrián J. Rovetto  
Quality Assurance Director

## Indicator Integrator

### For Ethylene Oxide Sterilization

IT10

#### Composition

Each integrator consists of a plastic holder and a glass capillary containing the indicating resin. Each indicator is inside a polyethylene bag which should not be changed or removed for integrator use.

#### Product description

Integron® IT10 indicator integrator is specifically designed to react with all critical parameters on ethylene oxide (EO) sterilization cycle (temperature, gas concentration, humidity and sterilization time).

The indicator integrator belongs to class 5 for sterilization chemical indicators. Integron® IT10 indicator integrators have a reference pointer on the label (orange triangle), which indicates the minimum acceptance limit. This means that the reference pointer indicates the minimum limit on color change for which the required critical parameter values are accepted to meet an inactivation status on a *Bacillus atrophaeus* ATCC 9372 spore population with D value specified between the limits accepted by the corresponding standards. The reference pointer corresponds to the kill time of Bionova® biological indicators with  $1.5 \times 10^6$  UFC population, measured by a resistometer.

#### Warning!

Do not use Integron® IT10 indicator integrator to control steam sterilization cycles, dry heat, chemical vapors, radiation or any other sterilization process different from Ethylene Oxide sterilization.

Do not re-use the integrator indicators.

Do not remove the indicator integrator from its corresponding bag.

Do not use indicator integrators with damaged bags.

Do not use Integron® IT10 indicator integrator to control EO sterilization processes for gas ampoule systems in polyethylene bags.

#### Important considerations

Class 5 Integron® IT10 Indicator Integrators permit to integrate critical parameters on EO sterilization and correspond them to the kill or inactivation of microorganism population with specific D value.

The reference pointer has been established under sterilization conditions where Bionova® biological indicator are negative.

It is advisable to use biological indicators along with indicator integrators in order to assure sterility.

#### Storage

Store in a dark place and at temperatures between 15-30 °C, 35-60 % relative humidity. Do not freeze.

Do not store near sterilizing agents or other chemical products.

Do not press glass capillary.

#### Directions for use

1. Identify Integron® IT10 indicator integrator by writing sterilizer number (in case it has more than one), load number and processing date on the bag.

2. Pack the indicator integrator along with the items for sterilization in an appropriate package according to recommended sterilization practices. Place the indicator integrator in those areas in which a priori you consider more inaccessible for sterilizing agent (EO).

3. Sterilize as usual.

4. After sterilization, evacuation and ventilation processes have finished you can:

A. Remove indicator integrator by following the required safety measures to avoid contact with remaining sterilizing agent.

B. Ventilate load along with indicator integrator.

5. Make the indicator integrator reading and record result.

Orange to blue color change up to reference pointer or beyond it indicates sterilization critical parameters values have been successfully met. This change is related to the kill of *Bacillus atrophaeus* spore population with D value on Bionova® biological indicators and to populations established under the corresponding standards.

**Warning!** Do not re-use sterilizer until color change on indicator integrator capillary has reached or gone beyond reference pointer and meets the acceptance area showed on the integrator label, and until biological indicators turn negative.

Integron® IT10 indicator integrators are manufactured according to EN ISO 11140-1: 2009 and IRAM 37101-1: 1999 standards.

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