Indicator Integrator

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Class 5 integrator indicator according to EN ISO 11140-1:2009.

Product description

Integrator® Terragene® IT26-C integrator indicators were developed for monitoring Steam sterilization processes between 118 °C and 138 °C and ensure an adequate control of the effectiveness of sterilization processes by monitoring all critical parameters of steam sterilization (temperature, time, steam quality). Integrator® Terragene® IT26-C integrator indicator consists of a special paper wick and a steam and temperature sensitive chemical pellet contained in a paper / film / foil laminate. The chemical pellet melts and migrates as a dark bar along the paper wick. The migration is visible through a zone marked ACCEPT or REJECT, thus indicating whether sterilization conditions were met. The extent of migration depends on steam quality, time and temperature. The ACCEPT result is reached when a theoretical spore population reaches its kill time; indicating integration condition has been reached. This condition is calibrated with the kill time of a 10⁶ *Geobacillus stearothermophilus* spores population, calculated in BIER (Biological Indicator Evaluator Resistometer).

According to EN ISO 11140-1:2009 standard, integrating indicators must have a response that correlates to the performance of the above mentioned biological indicator at three temperatures: 121 °C, 128 °C and 135 °C. In addition, the Stated Value at 121 °C should be greater than 16.5 minutes. These requirements ensure a proper performance of the integrator within the range of temperatures intended for use.

Characteristics

Integrator® Terragene® IT26-C integrator dimensions are 52 mm x 21 mm. Stated Values are listed in the table on the front of the package. The table lists the time in minutes at the corresponding temperature for chemical indicators in this lot to reach their endpoint when tested in a steam resistometer under highly controlled conditions with saturated steam.

Indications for use

Use the integrator indicator to monitor Steam sterilization processes between 118 °C and 138 °C.

Precautions

Do not use Integrator® Terragene® IT26-C integrator indicator to monitor Dry Heat, Ethylene Oxide, or any sterilization process other than Steam.

Instructions for use

- Carefully tear off or use scissors to open at the notch at the top of the package to make the initial opening. Remove only the number of IT26-C Integrator® Terragene® integrator indicators needed. Reseal the package by folding the open end over several times.
 - Place an Integrator® Terragene® IT26-C integrator indicator in each pack, peel pouch, container system or tray to be steam sterilized in the area determined to be the least accessible to steam penetration.
 - Process the load according to established procedures.
 - After processing, the dark bar should have reached the ACCEPT zone of Integrator® Terragene® IT26-C integrator indicator. If the dark bar has not reached the ACCEPT zone, a REJECT result is indicated and the items in the pack, peel pouch, container system, or tray were not exposed to proper steam sterilization conditions. These items should be reprocessed according to established procedures and a new IT26-C has to be used. In some extended or wet steam cycles, the dark bar can display discoloration in some zones of the reading window, imposing a difficulty for result's interpretation (see reference guide). Despite this, regarding cycle integration parameters, said process can be considered as Accepted.
- In case the IT26-C indicator shows a damp appearance after the sterilization process, we recommend planning an equipment's maintenance.

Storage and shelf life

- Store unopened and resealed packages at 30-80 % Relative Humidity condition at room temperature (10-30 °C). Store away from direct sunlight. Do not store near strong alkaline or acidic products such as cleaning/disinfecting agents.
- After use, the indicator will not change visually within 6 months when stored at above conditions.
- Integrator® Terragene® IT26-C integrator indicators contained in an unopened package have a 5-year shelf life from the date of manufacture when stored at recommended conditions. The expiration date is printed on the package label. Do not use indicators after their expiration date.

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Results reference guide

Accept
Regular CyclesAccept
Extended or wet Cycles

Reject

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Quality Certification

Code	Temp.	SV (Time)
LOT _____	121 °C	_____ min.
MW _____	128 °C	_____ min.
SA _____	135 °C	_____ min.

SV – stated value: Value of a critical process variable at which the indicator is designed to reach its endpoint as defined by the manufacturer.

According to EN ISO 11140-1:2009 standard

Lic. Adrián J. Rovetto
Technical director

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