MC70-80

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MC70-80

Culture medium with pH indicator.

For growth of *Geobacillus steraothermophilus* ATCC7953 bacterial spores.

Usage

To test growth of *Bacillus pumilus* ATCC 27142 bacterial spores used for monitoring Gamma Radiation sterilization processes.

Applicable Regulation

ISO11138-1:2006; IRAM 37102-1:1999.

Classification

Class 1, according to risk.

Authorization

Designed under Quality Management System standards ISO 13485:2003/NS-EN ISO 13485:2012. ANMAT (Argentinean National Administration of Drugs, Food and Medical Technology) PM 1614-1.

Characteristics

Sterile growth medium is specially formulated for use with our BIONOVA® products (also suitable for other brand products).

Growth medium color: red

Glass bottle: 143 mm high, 69 mm external diameter.

Medium volume: 80 ml.

For germination and growth of *Bacillus pumilus* ATCC 27142 bacterial spores.

Incubation conditions:

48 hours (maximum) at 37 ± 2 °C.

Medium turns to yellow when spores grow.

Environmental conditions during manufacture

15-30 °C room temperatures, RH 30-80 %. Medium is sterilized once the bottle is filled.

Storage conditions

T = 4-20 °C, RH 30-80 %, keep out of direct light.

Transportation conditions

Storage conditions should be strictly followed.

Transport in closed and reinforced boxes in order to avoid damages.

Transportation of this product does not represent a risk for health.

Shelf-life

24 months.

Packing

Screw capped bottle.

Information only on product's label.

Labelling

On product's box: product code and description, operating instructions, batch number, bacterial strain and expiration date.

Possible target markets

Healthcare and Industry.

Other important information (optional)

It is advisable to incubate at 37 °C in BIONOVA® IC10/20 incubator.

Note: When necessary, limits of each product's parameters should be specified.

Description of identified and/or predictable risks which could represent a risk for the product

Do not store the product near sterilizing agents.

Do not freeze.

The positive biological indicator can be autoclaved at 121 °C for at least 20 minutes, at 132 °C for at least 15 minutes in gravity displacement steam sterilization, or at 134 °C for at least 10 minutes in vacuum steam sterilizer.