



PCD Process Challenge Device

Test Pack with Class 5 moving front Integrator indicator and SCBI for monitoring dynamic-air-removal (pre-vacuum) and gravity displacement Steam autoclaves

121 °C gravity displacement steam sterilization cycles ≥ 30 minutes
132/134 °C dynamic-air-removal steam sterilization cycles ≥ 4 minutes

Instructions

- 1/ Place the pack, with this side up, on the bottom shelf near the door of a normally loaded steam autoclave and run sterilization cycle.
- 2/ Remove pack from autoclave. **Caution!** The steam test pack may be hot. Personnel should wear appropriate gloves and safety glasses.
- 3/ Check that the external process indicator color on the outside of the test pack has changed.
- 4/ Open the test pack and check IT26-C integrator indicator for correct exposure. If the dark bar has not reached the ACCEPT zone, this indicates the inside of the pack, and therefore the load, was not exposed to sufficient steam sterilization conditions. Check the sterilization process.
- 5/ Process SCBI according to instructions on the back. Fill out required information on the Record Card.
- 6/ Discard remainder of the test pack. It cannot be reused.

Turns to dark grey/black
Class 1 - EN ISO 11140-1