Chemical Indicators

Result Reference Guide

Turns to → Green

Chemdyne®  FORM  CD53

Unprocessed

Turns to → Green

Chemdyne®  FORM  CD53

Processed and satisfactory

Recommended Information

03-2017 / JP

Turns to → Green

08/324

Chemdyne®  FORM  CD53

11-2017

1

Sterilization date | Initials of the person in charge | Package content | Other information

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Explanation of symbols

FORM

Product designed for use in formaldehyde sterilization processes.

CE

CE mark.

EC REP

Authorized representative in the European Community.

MD

Medical Device.

LOT

Batch number.

⌚

Expiration date.

🏭

Manufacturer.

EN Chemical Indicators

For monitoring formaldehyde sterilization processes

Type 1 Indicators according to ISO 11140-1:2014

Product description
Chemdyne® CD53 Chemical Indicator has been specially developed to control formaldehyde sterilization processes. It consists of a double self-adhesive substrate of 29 x 22.5 mm printed with a highly sensitive innovative ink that turns from bordeaux-red to green when exposed to formaldehyde. Its automated record system allows to control the cycle by identifying the load, the sterilization date and, eventually, the sterilizer (in case of having more than one). Its self-adhesive properties offer the possibility of sticking it onto the load or any other material to be sterilized. Chemdyne® CD53 Chemical Indicator has been specially designed to adhere onto non-woven wrappings, medical grade paper, polyester and polypropylene. Chemdyne® CD53 Chemical Indicators can be used either with or without Chemdyne® CG3 Labeler. Using them with Chemdyne® CG3 Labeler enables not only to speed-up the process of placing them onto the surfaces but to record important information related to the sterilization process.

Instructions for use
When using Chemdyne® CD53 Chemical Indicator without Chemdyne® CG3 Labeler
1. Peel off Chemdyne® CD53 Chemical Indicator from the roll and stick it onto the package to be controlled.
2. Run a normal sterilization process.
3. Once the process has finished, open the sterilizer door, wait five minutes and remove the load. **CAUTION:** wear safety glasses and gloves when removing the sterilized package.
4. Check the final color reached by Chemdyne® CD53 Chemical Indicator. A green final color proves the chemical indicator has been exposed to formaldehyde. **IMPORTANT:** this color change does not fully evidence the process effectiveness to achieve sterility. If the final color does not change, check the sterilization process.

When using Chemdyne® CD53 Chemical Indicator with Chemdyne® CG3 Labeler
See Chemdyne® CG3 Labeler Instructions for Use.

Precautions
Do not use Chemdyne® CD53 Chemical Indicator to control sterilization processes different from formaldehyde.
Do not reuse Chemdyne® CD53 Chemical Indicator.
Do not wet.

Storage and shelf life
Store in a dry place, protected from light, at a temperature between 10-30 °C and at a relative humidity between 30-80 %.
Do not store near sterilizing agents.
Chemdyne® CD53 Chemical Indicators have an expiration date of 5 years from the date of manufacture if stored under recommended conditions. Do not use Chemdyne® CD53 Chemical Indicators after their expiration date. Endpoint stability reaction: chemical indicator endpoint shall remain unchanged for a period of 6 months when stored at previously indicated conditions.

Disposal
Discard chemical indicators after use according to your country's healthcare and safety regulations.

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Terragene S.A