Bionova® IC10/20FR Auto-reader

Symbols | Símbolos | Símbolos | Símboli

You may see one or more of these symbols on the packaging or labeling of this product. I Es possibile che vea uno o más de estos símbolos en el envase o el etiquetado de este producto. I Poderá ver um ou mais destes símbolos na embalagem ou rotulagem deste produto. I È possibile che uno o più di questi simboli siano presenti sulla confezione o sull'etichetta di questo prodotto.

	Humidity limitation Limite de umidade Limite di umidità
	Temperature limit Limite de temperatura Limite de temperatura Limite di temperatura
	Caution Precaución Precaução Attenzione
	Caution, hot surface Precaución, superficie caliente Attenzione: superficie calda Cuidado, superficie quente
	CE Mark Marca CE Marcado CE Marchio CE
	Direct current Corriente continua Corrente diretta Solo corrente continua
	For indoor use only Solo se debe usar en interiores Apenas para uso interior Solo per utilizzo all'interno
	Contains or presence of natural rubber latex Contiene o presenta látex de caucho natural Contiene o vi la presenza di lattice di gomma naturale Contém látex de borracha natural

Note: Do not use this product other than for the manner specified by Terragene S.A., otherwise the protection provided by the product might be affected.
Note: Only Terragene S.A. authorized personnel may access or service the internal components of the auto-reader. Parts or components inside the auto-reader should not be handled by the user.

Operating conditions		
Power supply specifications		
Input parameters	Operating conditions	Units
Voltage range	(100-240)	AC Volts
Frequency	50-60	Hertz
Current	1	Ampere
Output parameters	Values	Units
Voltage	12	DC Volts
Current	3	Amperes

Terragene S.A. recommends the use of Uninterruptable Power Supply (UPS) instead of voltage stabilizers, since they fulfill two functions: To stabilize and maintain the energy during a power outage.

Environmental operating conditions		
Environmental conditions	Operating conditions	Units
Altitude	3500 (max)	Meters
Operation temperature	10-30	Celsius
Relative humidity	30-80	%
Installation/Over-voltage	Category II	
Pollution degree	2	
Storage temperature	5-40	Celsius
Voltage	12	DC Volts

Device safety compliance
The Bionova IC10/20FR Auto-reader complies with the following standards and directives:

Electrical Safety	IEC 61010-1 IEC 61010-2-010 Low Voltage Directive 2014/35/EU
Electromagnetic Compatibility (EMC)	EN 61326-1 EMC Directive 2014/30/EU
European Commission	RoHS Directive 2011/65/EU WEEE Directive 2012/19/EU

The Bionova® IC10/20FR Auto-reader does not represent photobiological risk and does not generate dangerous optical radiation if used within normal operating conditions, as per the requirements of IEC 62471 standard.
Designed under Quality Management System standards ISO 13485:2016/EN ISO 13485:2016.
Note: The auto-reader has been evaluated for conformity for use in business environments. There may be concerns about radio interference if used in home environments. This applies exclusively to the Korean market due to KC certification requirements.

Cleaning and maintenance
Cleaning and decontamination of external surfaces
Disconnect the power supply cable and the USB cable from the auto-reader. If the device is hot, wait until it has cooled down before handling it.
Clean the external surfaces of the device using a microfibre cloth moistened with a solution of mild dish washing detergent and water. Wring out the cloth so it is damp but not dripping before cleaning, and wipe the outer surfaces of the auto-reader. Then, moisten a clean microfibre cloth with just water, and repeat the procedure until all traces of detergent are removed from the external surfaces of the device. After cleaning, allow the auto-reader to air dry for at least 1 hour before connecting the power supply cable or the USB cable again.
This cleaning procedure can be followed whenever considered appropriate.
Note: Do not pour liquid onto the device or immerse it in any of the external surfaces of the auto-reader.
If further cleaning is required, or if you have doubts about the cleaning agents you may use, please contact your local distributor.
Note: Do not clean the internal parts of the auto-reader.
Note: Do not pour liquid onto the device or immerse it in any liquid. Do not allow any liquid to enter the device while it is being cleaned.

Maintenance
The auto-reader does not require routine maintenance.

Warranty
The products are guaranteed to be free from material and workmanship defects when properly installed, maintained and used for their intended purpose as indicated in the applicable product label and/or the User Manual. The warranty only applies to the original purchaser.

Term
The warranty period for the Bionova® IC10/20FR Auto-reader is 1 (one) year from the date of installation, and may never be extended beyond 5 (five) years from the product's date of manufacture.

Limitation of liability
Terragene S.A. shall not be held liable for any loss or damage that result from the unsuitable use of the equipment, negligence or user's full responsibility.

Technical assistance
For information regarding the functioning or condition of the product, contact Terragene S.A. directly to info@terrargene.com or through our website www.terrargene.com. In USA, please call +1 844 837 7243.
Terragene S.A.
Ruta Nacional N° 9, Km 280 – CP 2130.
Parque Industrial Micropi – Alvear - Santa Fe - Argentina.

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EN Instructions for use

Note: Remember to set the auto-reader to your local date and time before starting incubations.

Setting up your auto-reader

Place the auto-reader on a firm surface, free from vibrations, away from direct sunlight, hot or cold air flows, chemical and corrosive or flammable substances. Do not place the auto-reader in a way that would make it difficult to remove the power supply's AC plug from mains. Leave a space of at least 10 cm from the auto-reader to the closest wall. Do not move the auto-reader periodically or while it is being used. Connect the auto-reader to a secure and stable electrical mains outlet socket.

Note: Do not wet or heat the auto-reader. If liquid is spilled on the auto-reader, disconnect it and clean it. For more information, please refer to the User Manual.

Using your auto-reader

1) Switch on the auto-reader by connecting the power supply's AC plug to mains and then connect the plug at the other end of power supply to the rear of the auto-reader (See image 1).

Note: Before switching it on, check that all incubation positions are empty.

2) Set the auto-reader local date and time. For more information, please refer to the User Manual.

3) Press and hold the incubation temperature selection button for 3 seconds (See image 2). Press again to select the desired temperature. After 4 seconds the temperature modification will be accepted automatically. Wait until the temperature & stability indicator light stops blinking.

Note: Changing the incubation temperature can only be done as long as there is not an ongoing reading.

4.A) SCBI Incubation procedure

4.A.1) Select the appropriate incubation time for the incubation position you are planning to use, according to the SCBI's Instructions for Use. To select the incubation time for incubation positions 1 to 6, press the left incubation time selection button. To select the incubation time for incubation positions 7 to 12, press the right incubation time selection button (See image 3.A).

4.A.2) Place an SCBI in an incubation position. The reading process will start automatically after placing the SCBI (See image 3.B).

4.B) PRO1 Indicator incubation procedure

4.B.1) To enable the PRO1 incubation position, press the following button for 1 second: **Pro** . (See image 4.A).

4.B.2) Place the PRO1 Indicator fully inside the PRO1 incubation position. Start the incubation and readout process by pressing the button: **Pro** .

5) Wait until a readout result has been carried out. The positive or negative indicator light will turn on to indicate the result and a ticket will be printed. For PRO1 indicators, the auto-reader will inform the equivalent of protein quantity using a BSA (Bovine Serum Albumin) calibration curve. Do not move, remove or change the position of the SCBI / PRO1 Indicator once the reading process has begun. If you do this, the results may be invalidated.

Note: To select a protein quantification threshold mode and a threshold value, please refer to the User Manual.

Disposal

Dispose of the SCBIs / PRO1 Indicators in accordance with your country's waste-disposal guidelines.

Connectivity

To connect your auto-reader to a personal computer (PC) by USB, connect one end of the USB cable provided to the rear of the auto-reader and the other end to a USB port on your PC. Connect the auto-reader to a PC to use the Bionova® Cloud Environment. For more information, please refer to the User Manual.

Bionova® IC10/20FR Auto-reader incubation programs

The following incubation programs are available for your auto-reader:

Temperature selection	Time selection	Incubation program
60 °C	20 minutes	20 minutes at 60 °C
	30 minutes	30 minutes at 60 °C
	1 hour	1 hour at 60 °C
	2 hours	2 hours at 60 °C
	3 hours	3 hours at 60 °C
	PRO (7 minutes)	PRO at 60 °C
	4 hours	4 hours at 37 °C
37 °C		

Note: Please read, understand and follow the SCBI / PRO1 Indicator Instructions for Use before using it.

Troubleshooting

Refer to the User Manual for detailed troubleshooting guidance.

Please visit www.terrargene.com/IC1020FR or scan the QR code to access the auto-reader User Manual.

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