QUICKSTART GUIDE

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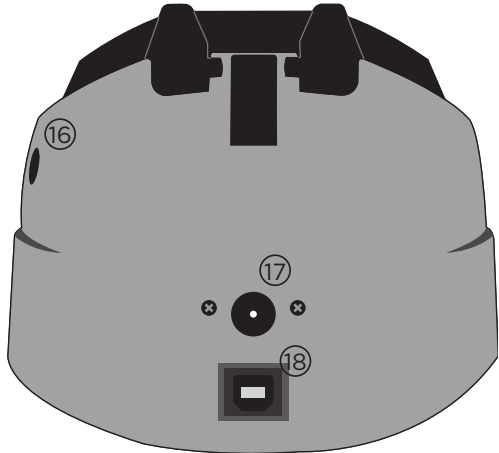
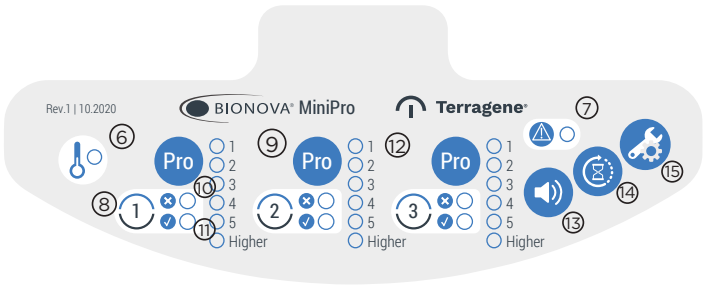
BIONOVA®
MiniPro

Terragene®

R04 | 05.2026

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EN Instructions for use

Setting up your auto-reader

Place the auto-reader on a firm surface, free from vibrations, away from direct sunlight, hot or cold air flows, chemical and corrosive or flammable substances. Do not place the auto-reader in a way that would make it difficult to remove the power supply's AC plug from mains. Leave a space of at least 10 cm between the auto-reader and the closest wall. Do not move the auto-reader periodically or while it is being used. Connect the auto-reader to a secure and stable electrical mains outlet socket. Do not wet or heat the auto-reader. If liquid is spilled on the device, disconnect it and clean it. Before turning it on, check that all reading positions are empty.

Using your auto-reader

- Switch on your auto-reader by connecting the power supply's AC plug to mains and then connect the plug at the other end of power supply to the rear of the auto-reader. (See image 1)
- Wait until the temperature stability indicator stops blinking, indicating that the auto-reader has reached a steady working temperature. Press and hold the button of the chosen position for 1 second to enable the position's readout mode. (See image 2)
- Wait until the quantitative protein concentration indicator stops blinking. Place the Protein Pen in the corresponding position. Press the PRO button again for 1 second to start the incubation. (See image 3)
- Wait until the incubation has finished. The green light will turn on in case of a negative result. The red light will turn on in case of a positive result. If a positive result is detected, press the alarm cancellation button to cancel the alarm. Do not remove the indicator until the ticket with the result of the incubation is printed. (See image 4)

Disposal: Dispose the Protein Pens in accordance with your country's waste-disposal guidelines.

Connectivity

To connect your auto-reader to a personal computer by USB, connect one end of the USB cable provided to the rear part of the auto-reader and the other end to the USB port on your PC. Connect the auto-reader to a PC to use the Bionova® Cloud Environment. Please refer to the User Manual for more information.

Product compatibility and incubation program

The following incubation program is available for your auto-reader:

Temperature	Time	Incubation program
60 °C	4 minutes	PRO1 (4 minutes at 60 °C)

Note: Please read and follow the Protein Pen Instructions for Use before its incubation.

Troubleshooting

Refer to the User Manual for a detailed troubleshooting guidance. Visit www.terragene.com/MINIPRO or scan the QR code to access the auto-reader User Manual.

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