Infection Control Division | Chemical Indicators

CD29 Multivariable Chemical Indicator

For monitoring Steam sterilization processes

Usage

United States

Terragene Chemdye® CD29 is a chemical process indicator intended for monitoring the efficacy of steam sterilization processes. The chemical indicator changes from yellow to dark brown/black to indicate that the conditions of the cycle have been met.

For gravity-air displacement Steam sterilization cycles: 121 °C, 30 minutes // 132 °C, 15 minutes // 132 °C, 25 minutes // 135 °C, 10 minutes.

For dynamic-air-removal Steam sterilization cycles: 132 °C, 4 minutes // 135 °C, 3 minutes.

Outside the United States

Monitoring the efficacy of Steam sterilization processes.

Pass conditions: 121 °C, 15 minutes // 134 °C, 3.5 minutes.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN, ISO 13485:2016.

ISO 11140-1:2014.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

Classification

Class 1, according to risk (ANMAT).

FDA 510(k)

K191021.

Characteristics

Type 4 Multivariable Chemical Indicator (according to ISO 11140-1).

One-side laminated punched double card strips printed with indicator ink.

Size: 200 mm x 15 mm.

Initial color: **yellow**.

Final color: **black**.

100 % Toxic Heavy Metal free

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %, keep out of direct light.

Infection Control Division | Chemical Indicators

CD29 Multivariable Chemical Indicator

For monitoring Steam sterilization processes

Transport conditions

It is recommended to store it away from sunlight and at a temperature between 10 - 30 °C.

Transport in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent a risk for health.

Acceptable excursions during transport are as follows:

T = 2-10 °C and T = 30-37 °C for no longer than seven days. No restrictions for relative humidity.

Shelf life

5 years.

Packing

250 double strips per aluminum foil bag.

Packing information: product description, instructions for use, regulation, storage conditions, manufacturer information and data on label.

Labelling

On product's packing: product code and description, sterilization process for which the product was designed, classification according to regulation, color change, presentation, batch number, expiration date, bar code and datamatrix code.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information

Read product's instructions for use thoroughly before use.

Precautions

Do not store the product near sterilizing agents.

Do not expose this product to EO, Dry Heat or any sterilization processes other than Steam.

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