Infection Control Division | Chemical Indicators

CD42 Process Indicator

For monitoring Plasma or Vaporized Hydrogen Peroxide sterilization processes

Usage

United States

Terragene® Chemdye® Chemical Process Indicator (CD42) is a chemical process indicator intended for monitoring the efficacy of vaporized hydrogen peroxide sterilization processes in the following systems:

Vaporized Hydrogen Peroxide
Cycles
Sterrad® 100S Sterilization System
Sterrad® NX Sterilization System (Standard and Advanced cycles)
Sterrad® NX with ALLClear® Technology Sterilization System (Standard and Advanced cycles)
Sterrad® 100NX Sterilization System (Standard, Flex, Express and Duo cycles)
Sterrad® 100NX with ALLClear® Technology Sterilization System (Standard, Flex, Express and Duo cycles)
V-Pro® maX Low Temperature Sterilization System (Non Lumen, Lumen and Flexible cycles)
V-Pro® maX 2 Low Temperature Sterilization System (Fast Non Lumen, Non Lumen, Lumen and Flexible cycles)
Sterizone® VP4 Sterilizer (Cycle 1)

The chemical indicator changes from red to yellow to indicate that the conditions of the cycle have been met.

Outside the United States

Terragene® Chemdye® CD42 Chemical Process Indicator has been designed for monitoring vaporized hydrogen peroxide sterilization processes.

Endpoint conditions: 50 °C, 6 minutes, 2.3 mg/l H₂O₂.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN, ISO 13485:2016. ISO 11140-1:2014.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

Classification

Class 1, according to risk (ANMAT).

FDA 510(k)

K191021; K232103.

Infection Control Division | Chemical Indicators

CD42 Process Indicator

For monitoring Plasma or Vaporized Hydrogen Peroxide sterilization processes

Characteristics

Type 1 Chemical Indicator (according to ISO 11140-1).

Strips printed with indicator ink.

Size: 105 x 18 mm (approximate).

Initial color: **red**.

Final color: **yellow**.

100 % Toxic Heavy Metal free

To interpret results visually, use the reference guide found in the Indications for Use.

Optionally, the readout of the chemical indicator can be done using Trazanto Automatic Quality Control and Traceability System (see instructions for use in device's manual).

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %, keep in a dark place in its original box.

Transport conditions

It is recommended to store it away from sunlight and at a temperature between 10-30 °C.

Transport in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent a risk for health.

Acceptable excursions during transport are as follows:

T = 2-10 °C and T = 30-37 °C for no longer than seven days. No restrictions for relative humidity.

Shelf life

5 years.

Packing

250 or 500 strips per aluminum foil bag.

Packing information: product description, instructions for use, storage conditions and manufacturer information.

Labelling

On product's packing: product code and description, process for intended use, classification according to regulation, color change, product presentation, batch number, expiration date, barcode and datamatrix code.

Infection Control Division | Chemical Indicators

CD42 Process Indicator

For monitoring Plasma or Vaporized Hydrogen Peroxide sterilization processes

Possible target markets

Healthcare, Food, Pharmaceutical and Medical industries.

Other important information

Read product's instructions for use thoroughly before use.

Precautions

Do not store the product near sterilizing agents.

Do not expose this product to steam, dry heat or any sterilization processes other than H₂O₂.

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