Infection Control Division | **Cleaning Indicators**

CDWU-Z Ultrasonic Cleaning Indicator

For ultrasonic cavitation performance tests.

Usage

Designed for ultrasonic cleaning processes' routine control and monitoring ultrasonic cavitation effectiveness of ultrasonic washing machines operating at frequencies equal to or greater than 35 kHz, at temperatures ranging from 18 °C to 70 °C.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN ISO 13485:2016.

Characteristics

2 ml clear glass vial: 31.0 mm high x 11.0 mm external diameter, 8.0 mm neck diameter. Wall thickness: 0.5 mm.

Brown/Red rubber septum: 8.0 x 1.0 mm.

Aluminum top cap: 5.5 mm high x 11.7 mm diameter. Wall thickness: 0.3 mm.

Ceramic cylinders: 2.5 mm high and 2.5 mm diameter.

Blue clear reactive solution: 1.5 ml.

Allows detection of deficiencies in the cavitation performance of the ultrasonic washing machine.

A self-adhesive Record Card accompanies each vial where data about the ultrasonic washing control process can be recorded.

Challenge level: high.

Initial Color: **blue**.

Result: **yellow transparent solution**.

NOTE: If unfavorable results are obtained, check working frequency of the ultrasonic cleaner, time and temperature of the ultrasonic washing, presence of bubbles within the wash solution, location of indicator and general condition of ultrasonic washing machine.

Environmental conditions during manufacture

T = 10-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %, keep in a dark place in its original box.

Transport conditions

It is recommended to keep it away from sunlight and at a temperature between 10-30 °C.

Transport in closed and strengthened boxes in order to avoid damages. Transportation of this product does not represent a risk for health.

Infection Control Division | **Cleaning Indicators**

CDWU-Z Ultrasonic Cleaning Indicator

For ultrasonic cavitation performance tests.

**Shelf life**

18 months.

Packing

6 or 30 units per box.

Packing information: product code and description, process for intended use, presentation, storage conditions, manufacturer information and data on label.

Labelling

On product's label: product's code and description, product's presentation, batch number, expiration date, bar code and datamatrix code.

Possible target markets

Healthcare and Industries.

Other important information

Read product's instructions for use thoroughly before use.

Precautions

Do not re-use the CDWU-Z indicator.

Do not open the indicator before its use, since this will cause its deterioration.

Do not use the CDWU-Z indicator for monitoring normal washing using non-ultrasonic cleaners.

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