Infection Control Division | **Chemical Indicators**

CT10 Indicator tape

For ethylene oxide sterilization processes

Usage

Cintape® CT10 is a chemical process indicator tape for external use designed for monitoring ethylene oxide (EO) sterilization processes. Conditions: 600 mg/l EO, 60 % RH, 54 °C, 20 minutes // 600 mg/l EO, 60 % RH, 37 °C, 25 minutes.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/EN ISO 13485:2016.
ISO 11140-1:2014.

Authorization

ANMAT (Argentinian National Administration of Drugs, Food and Medical Devices) PM 1614-4.

Classification

Class 1, according to risk (ANMAT).

Characteristics

Type 1 Chemical Indicator.
Roll of self-adhesive tape printed with indicator ink.
Size: 50 m x 18 mm.
Initial color: orange.
Final color: green.
Indicators 100 % Lead Free

Lead free

Tape surface treated with covering materials that allow an easy and continuous writing. The tape is capable of adhering to wrap cloth, paper and plastic. Designed to ensure that the tape remains adhered to the surface after sterilization and leaves no trace of adhesive on the surface of materials after removal.

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %, keep out of direct light.

Transport conditions

Storage conditions should be strictly followed.
Transport in closed and reinforced boxes in order to avoid damages.
The transport of this product does not represent a risk for health.

Shelf life

5 years.

Packing

1 roll of 50 m x 18 mm.
Packing information: product description, classification according to regulation, storage conditions, manufacturer information and data on packing's label.

Infection Control Division | **Chemical Indicators**

CT10 Indicator tape

For ethylene oxide sterilization processes

Labeling

On the product: process for intended use, classification according to regulation, batch number, expiration date and manufacturer.

On product's packing: product code and description, process for intended use, conditions of use, classification according to regulation, color change, tape dimensions, batch number, manufacture and expiration date, bar code and datamatrix code.

Possible target markets

Healthcare, Food, Pharmaceutical and Medical Industries.

Precautions

Do not store the product near sterilizing agents.

Do not expose this product to steam, dry heat or any sterilization processes other than EO.

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