Infection Control Division | **Chemical Indicators**

CT40 Indicator Tape

For Plasma or Vaporized Hydrogen Peroxide sterilization processes

Usage

United States

Terragene® Cintape® Chemical Process Indicator Tape (CT40) is a chemical process indicator tape intended for monitoring the efficacy of vaporized hydrogen peroxide sterilization processes in the following systems:

Vaporized Hydrogen Peroxide
Cycles
Sterrad® 100S Sterilization System
Sterrad® NX Sterilization System (Standard and Advance cycles)
Sterrad® NX with ALLClear® Technology Sterilization System (Standard and Advanced cycles)
Sterrad® 100NX Sterilization System (Standard, Flex, Express and Duo cycles)
Sterrad® 100NX with ALLClear® Technology Sterilization System (Standard, Flex, Express and Duo cycles)
V-Pro® maX Low Temperature Sterilization System (Non Lumen, Lumen and Flexible cycles)
V-Pro® maX 2 Low Temperature Sterilization System (Fast Non Lumen, Non Lumen, Lumen and Flexible cycles)
Sterizone® VP4 Sterilizer (Cycle 1)

The indicating tape changes from purple to green when exposure to vaporized hydrogen peroxide.

Outside the United States

Terragene® Cintape® CT40 is a chemical process indicator tape for external use for monitoring vaporized hydrogen peroxide sterilization processes.

Endpoint conditions: 6 minutes, 50 °C, 2,3 mg/l H₂O₂.

Applicable regulation

Designed under Quality Management System standards ISO 13485:2016/NS-EN, ISO 13485:2016. ISO 11140-1:2014.

Authorization

ANMAT (Argentinean National Administration of Drugs, Food and Medical Devices) PM 1614-4.

Classification

Class 1, according to risk (ANMAT).

FDA 510(k)

K191021; K232103.

Infection Control Division | **Chemical Indicators**

CT40 Indicator Tape

For Plasma or Vaporized Hydrogen Peroxide sterilization processes

Characteristics

Type 1 Chemical Indicator (according to ISO 11140-1).

Roll of self-adhesive tape printed with indicator ink.

Size: 50 m x 20 mm.

Initial color: **purple**.

Final color: **green**.

100 % toxic heavy metal free.

Adhesive capable of adhering to wrap cloth, paper and plastic. Designed with advanced technologies to ensure that the tape remains adhered to the surface after sterilization and leaves no trace of adhesive on the surface of the materials after removal.

Environmental conditions during manufacture

T = 15-30 °C, RH = 30-80 %.

Storage conditions

T = 10-30 °C, RH = 30-80 %, keep out of direct light.

Transport conditions

It is recommended to store it away from sunlight and at a temperature between 10-30 °C.

Products should be transported in closed and reinforced boxes in order to avoid damages.

The transport of this product does not represent a risk for health.

Acceptable excursions during transport are as follows:

T = 2-10 °C and T = 30-37 °C for no longer than seven days. No restrictions for relative humidity.

Shelf life

5 years.

Packing

1 roll of 50 m x 20 mm.

Packing information: product description, regulation, storage conditions, presentation, manufacturer information and data on packing's label.

Labeling

On roll's cone: process for intended used, classification according to regulation, batch number, expiration date and manufacturer.

On product's packing: product code and description, process for intended used, classification according to regulation, color change, tape dimensions, batch number, manufacture and expiration date, bar code and datamatrix code.

Infection Control Division | **Chemical Indicators**

CT40 Indicator Tape

For Plasma or Vaporized Hydrogen Peroxide sterilization processes

Possible target markets

Healthcare, Food, Pharmaceutical and Medical Industries.

Other relevant information

Read product's instructions for use thoroughly before use.

Precautions

Do not store the product near sterilizing agents.

Do not expose this product to ethylene oxide (EO), steam or any sterilization processes other than H₂O₂.

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