Infection Control Division | Accessories

Bionova® IC10/20FR Auto-reader Incubator

For incubation and readout of Rapid, Super Rapid or Ultra Rapid Biological Indicators and Hygiene Monitoring Systems

Usage

United States

Terragene® Bionova® IC10/20FR Auto-reader incubates at 60 °C and reads the Terragene® Bionova® SCBIs (BT220 and BT222) for fluorescence results at 1 hour (BT222) and 3 hours (BT220).

Outside the United States

For incubation and readout of:

Bionova® rapid fluorescence readout biological indicators (BIs):

-BT220 (steam): 3 h at 60 °C.

-BT110 (ethylene oxide): 4 h at 37 °C.

-BT102 (formaldehyde): 2 h at 60 °C.

Bionova® super rapid fluorescence readout BIs:

-BT222 (steam): 1 h at 60 °C.

-BT96 (VH2O2): 30 min. at 60 °C.

-BT97 (VH2O2): 1 h at 60 °C.

Bionova® ultra rapid fluorescence readout BIs:

-BT224 (steam): 20 min. at 60 °C.

Chemdye® hygiene monitoring systems:

-PRO1 MICRO: 7 min. at 60 °C.

-PRO1 ENDO: 7 min. at 60 °C.

Applicable regulation

Designed under Quality Management System standards
ISO 13485:2016/ EN ISO 13485:2016.

IEC 61010-1

IEC 61010-2-010

Low Voltage Directive 2014/35/EU.

EN IEC 61326-1

EMC Directive 2014/30/EU.

RoHS Directive 2011/65/EU.

WEEE Directive 2012/19/EU

FDA 510(k)

K163646.

Characteristics

12 positions (38 mm deep, 9 mm diameter) to incubate rapid, super rapid and ultra rapid fluorescence readout self-contained biological indicators and 1 position (38 mm deep, 10 mm diameter) to incubate the pen of the hygiene monitoring systems.

1 ampoule crusher.

Special hole for external thermometer.

Dual temperature system: allows selecting between two different incubation temperatures (37 and 60 °C). Allows running two different incubation times simultaneously, provided they are of 60 °C.

Sound alarm for event indication.

USB connection for recording results on PC using Bionova® Cloud Web Application.

It stores the last 13 results.

Thermal printer of results. Allows to reprint the last 13 results.

Automatic detection and cancellation of biological indicator fluorescence reading.

Firmware update by the user.

Fluorescence reading wavelength: ~ 460 nm.

Infection Control Division | Accessories

Bionova® IC10/20FR Auto-reader Incubator

For incubation and readout of Rapid, Super Rapid or Ultra Rapid Biological Indicators and Hygiene Monitoring Systems

Dimensions: 182 mm high, 257 mm diameter, 286 mm depth.
Voltage range: 100-240 VAC.
Power: 36 W.
Frequency: 50-60 Hz.
Operating temperature: 10-30 °C.

Storage conditions

T = 5-40 °C, RH = 30-80 %, keep away from direct sunlight.

Transport conditions

It is recommended to respect the storage conditions during transport. Products should be transported in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent any risk for human health.

Expected lifetime

5 years.

Packing

1 unit per box.

Packing information: product code and description, regulatory data and manufacturer information.

Labelling

On product: product code and description, operating conditions, batch and serial number, Data Matrix, manufacture date, manufactory firmware and manufacturer data.

On product's box: product code and description, storage conditions, batch and serial number, Data Matrix, manufacture date and manufactory firmware.

Possible target markets

Healthcare, Food, Pharmaceutical, Dental and Medical Devices Industries

Other important information

IC10/20FR auto-reader has a button located on the incubator panel to select temperatures at 37 °C or 60 °C. Furthermore, it also has a button which allows setting incubation time for half of the positions (1 to 6), and another button to select the incubation time for the remaining positions (7 to 12), provided that the incubation temperature is 60 °C.

IC10/20FR auto-reader has an automatic readout system and a printer to record the results.

Precautions

Do not pour any liquid inside.

Do not immerse in any liquid.

Only for indoor use.

Disconnect the power cord before cleaning.

Do not use abrasive or corrosive cleaners or disinfectants.

Make sure that the auto-reader is connected to a properly rated power cord.

This product contains dry natural rubber.

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