Infection Control Division | Accessories

Bionova® IC10/20FRLCD Auto-reader Incubator

For incubation and readout of Rapid, Super Rapid or Ultra Rapid Biological Indicators and Hygiene Monitoring Systems

Usage

United States

Terragene® Bionova® Reader Incubators IC10/20FRLCD incubate at 60 °C and 37 °C and read the Terragene® Bionova® SCBIs for fluorescence results at the times prescribed in the User Manual.

Outside the United States

For incubation and readout of:

Bionova® rapid fluorescence readout biological indicators (BIs):

-BT220 (steam): 3 h at 60 °C.

-BT110 (ethylene oxide): 4 h at 37 °C.

-BT102 (formaldehyde): 2 h at 60 °C.

Bionova® super rapid fluorescence readout BIs:

-BT222 (steam): 1 h at 60 °C.

-BT96 (VH2O2): 30 min. at 60 °C.

-BT97 (VH2O2): 1 h at 60 °C.

Bionova® ultra rapid fluorescence readout BIs:

-BT224 (steam): 20 min. at 60 °C.

Chemdye® hygiene monitoring systems:

-PRO1 MICRO: 7 min. at 60 °C.

-PRO1 ENDO: 7 min. at 60 °C.

Applicable regulation

Designed under Quality Management System standards
ISO 13485:2016/ EN ISO 13485:2016.

IEC 61010-1

IEC 61010-2-010

Low Voltage Directive 2014/35/EU.

EN IEC 61326-1

EMCDirective 2014/30/EU.

RoHS Directive 2011/65/EU.

WEEE Directive 2012/19/EU

FDA 510(k)

K191021.

Characteristics

12 positions (38 mm deep, 9 mm diameter) to incubate rapid, super rapid and ultra rapid fluorescence readout SCBI and 1 position (38 mm deep, 10 mm diameter) to incubate the pen of the hygiene monitoring systems.

1 ampoule crusher.

Special hole for external thermometer.

3.5" touchscreen LCD.

Temperature display on the screen and remaining time for each position.

Dual temperature system: allows selecting between two different incubation temperatures (37 and 60 °C). Allows running different incubation times simultaneously, provided they are of equal incubation temperature.

Sound alarm for event indication.

ETHERNET connection for recording results on PC using Bionova® Cloud Web Application.

It stores the last 208 results.

Thermal printer of results. Allows to reprint any of the last 208 results.

Infection Control Division | Accessories

Bionova® IC10/20FRLCD Auto-reader Incubator

For incubation and readout of Rapid, Super Rapid or Ultra Rapid Biological Indicators and Hygiene Monitoring Systems

Automatic detection and cancellation of biological indicator fluorescence reading.

Firmware update by the user.

Fluorescence reading wavelength: ~460 nm.

Dimensions: 180 mm high, 260 mm diameter.

Voltage range: 100-240 VAC.

Power: 36 W.

Frequency: 50-60 Hz.

Operating temperature: 10-30 °C.

Storage conditions

T = 5-40 °C, RH = 30-80 %, keep away from direct sunlight.

Transport conditions

It is recommended to respect the storage conditions during transport. Products should be transported in closed and reinforced boxes in order to avoid damages. The transport of this product does not represent any risk for human health.

Expected lifetime

5 years.

Packing

1 unit per box.

Packing information: product code and description, regulatory data and manufacturer information.

Labelling

On product: product code and description, operating conditions, batch and serial number, Data Matrix, manufacture date, manufactory firmware and manufacturer data.

On product's box: product code and description, storage conditions, batch and serial number, Data Matrix, manufacture date and manufactory firmware.

Possible target markets

Healthcare, Food, Pharmaceutical, Dental and Medical Devices Industries

Other important information

IC10/20FRLCD auto-reader has an automatic readout system and a printer to record data. It also has a history of records to query the last 208 results, which can be viewed from the device screen by accessing the auto-reader internal website.

Precautions

Do not pour any liquid inside.

Do not immerse in any liquid.

Only for indoor use.

Disconnect the power cord before cleaning.

Do not use abrasive or corrosive cleaners or disinfectants.

Make sure that the auto-reader is connected to a properly rated power cord.

This product contains dry natural rubber.

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